

ENZYCIDE[®]

Manufactured in GMP unit

Made in India

Product of
Indigenous
Research
& Inputs

Other Range

Infectrol[®] 2.45

Ultrabasil[®] Plus

Aldefree

Ultrabasil[®] 3Q

Envishield[®]

Microbat[®]



UPS Hygienes Pvt. Ltd.

(AN ISO 9001-2015 Certified Company)
Customer Care No.: 022-65214567 / 9821338361
E-mail: info@upshygiene.com

Your Partner In Infection Control

Blood protein / serum remover.

Multi Enzymatic Cleaner.

Recommended for Medical & Surgical Instruments, Endoscopes.

Composition

Combination of live enzymes
in detergent base

Packing

100ml	60 bott. / carton
1 ltr	10 cans / carton

Enzycide is a combination of different types of enzymes.

These enzymes break down protein in the blood & help cleaning. Ideal for processing difficult to clean instruments e.g. Micro instruments, Vascular instruments. Also useful for Surgical instruments of Stainless-Steel, Glass Tubing, glass Ware.

Enzycide is very easy to use. Dilute as per chart, depending upon soiling.

Enzycide is highly effective against proteins, carbohydrates & lipids.

Enzycide is cost effective. **Enzycide** is only better option to conventional detergents.

Use **Enzycide** for cold water soaking in Ultrasonic cleaners and also for sterilisers.

Enzycide does not form film coating, there by easy to rinse. Gentle for Endoscopes.

Use Enzycide in following dilutions, as recommended in chart, before disinfection.

Degree of Contamination	Dilutions	Soaking time
Light	5 ml. per ltr.	5 minutes
Moderate	7 ml. per ltr.	8 minutes
Heavy	10ml. per ltr.	8 minutes
For Ultrasonic Bath	7 ml. per ltr.	2 minutes

Rinse thoroughly after soaking.

Instruments cleaned with Enzycide are suggested to be disinfected or sterilised with infectrol 2.45 (2.45 % of glutaraldehyde alkaline solution with activator) or Infectrol GF.

Handle with care. Use gloves while handling.

Enzycide is low foaming, making ideal choice for ultrasonic cleaning.

- Keep away from eye, avoid contact with body & clothes.
- Store at temperature between 25°C to 30°C

Website : www.upshygiene.co.in

Helpline : 022-65214567 / 9821338361

